

Able Spacer

The only Anti-Viral / Bacterial Valved holding Chamber available, uniquely facilitating the use of pMDI medication

- + Anti-viral / Microbial
- + Effective against Covid-19
- + Positive whistle mask available.
- + Works with free Rafi-Tone App



Anti-Viral & Anti-Bacterial

The only anti-viral and anti-bacterial Spacer range available. It's silver ion technology inhibits microbial growth, and is shown to be effective against COVID-19.

210ml chamber capacity

Easy use size. Also available in a collapsible version (A2A).

Plume shaped chamber

Helps to minimise drug loss within the spacer.

Easy actuation one-way valve

Uniquely manufactured using silicone, allowing easy movement for paediatric or vulnerable patients with low inspiratory flow rates.

Available with or without mask

Provides versatile use for both adults and children.



**Clement
Clarke**
International

Able Spacer

Able Spacer is designed to help patients with poor inhaler co-ordination use their pMDI. The device and masks are made from low-static transparent plastic, incorporating anti-bacterial/microbial silver ion technology, which helps protect against contamination and inhibits microbial growth. Able Spacer features a one-way valve, to prevent exhalation into the device and operates from less than 10 L/min meaning it can be used with a mask from birth. Able Spacer can be used with Positive Whistle Mask, which enables the use of the Rafi-Tone app.

Cleaning and Maintenance

Wash the spacer at least once a week:

- Remove spacer cap and separate two halves of chamber.
- Do not separate one-way valve from mouthpiece section.
- Wash the parts in warm soapy water.

Allow the parts to air dry naturally.

Re-assemble when completely dry.

Do not clean the device with a brush, cloth, or any other implement, as this will damage the device causing loose diaphragms which could enter the mouth and cause discomfort. Do not sterilise; extreme temperatures damage the device and could affect the medication dose.

Not suitable for dishwasher use; dishwasher detergents and extreme temperatures damage the device and cause loose diaphragms which could enter the mouth and cause discomfort.

Technical Specifications

	DEVICE	BOXED	BOXED WITH MASK
Dimensions (mm)	155 x 55 x 66	158 x 60 x 70	165 x 78 x 88
Weight (g)	62	76	138
Volume (ml)	210		
Resistance to flow (L/min)	10mm of water @ 30		
Product shelf life (Years)	5		
Lifetime (Months)	12		
Re-usable claim	Single Patient Re-usable		

PART	PIP CODE	DESCRIPTION	DRUG TARRIF PRICE*	QTY
3609120	266-0496	Able Spacer (boxed)	£4.39	1
3609121	353-0680	Able Spacer with small whistle mask (boxed)	£7.16	1
3609122	353-0672	Able Spacer with medium whistle mask (boxed)	£7.16	1
3609123	395-6497	Able Spacer with large mask (boxed)		1
3609150	266-0504	Small mask		1
3609151	266-0512	Medium mask		1
3609152	266-0520	Large mask		1
36091482		Small Whistle mask		50
36091472		Medium Whistle mask		50

*Prices correct as of latest Drug Tariff

Clement Clarke International Ltd

Cynon Valley Business Park • Mountain Ash • CF45 4ER • UK
+44 (0)1443 804999 • resp@clement-clarke.com • www.clement-clarke.com

